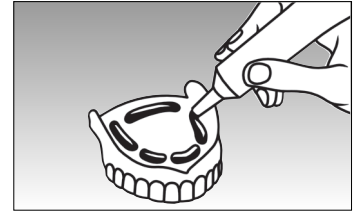
Adhesive Cream Extra-Strong

Crema adhesiva Extra-Fuerte

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GB **Protifix** Adhesive Cream

Instructions for use. Please read carefully!

Application: **Protifix**® Adhesive Cream Extra-Strong for dentures.

Dosage and mode of application: Daily dosage: up to 0.8 g adhesive cream. Clean the dentures of any residue and dab **Protifix**® Adhesive Cream Extra-Strong on the wet denture (in the upper or lower denture recess that goes next to the jaw). Press the tube in the lower third near the opening. If necessary, you may also apply a line of **Protifix**® Adhesive Cream Extra-Strong to the back of the upper plate. You should apply **Protifix**® Adhesive Cream Extra-Strong sparingly because any excess can impair the adhesion of your denture. Then fit the denture and hold it firmly in place for a few seconds. Wait 5 minutes before eating. The adhesive cream does not noticeably change the position of the denture. **After applying the cream, seal the tube carefully, taking care to ensure that the opening remains dry and clean because any ingress of liquid may cause the adhesive cream to harden.** Remove any residue of adhesive cream with a cotton-wool ball soaked in edible oil. Under certain circumstances oil may appear, which has no effect whatsoever on the creams' particularly good adhesive force. Please just remove the oil from the dentures. The use of adhesive cream should only be temporary.

Contraindications: Hypersensitivity to one of the ingredients. **Side effects:** In isolated cases, hypersensitivity reactions in the mucous membranes of the mouth or even systemically cannot be ruled out. If you suspect the presence of a hypersensitivity reaction, please contact your dentist. **Please note the following important advice:** To be used only on healthy mucous membranes. Keep out of reach of children. Do not use after the expiry date.

Composition: Mixture of sodium and calcium salts of the copolymer of methyl vinyl ether and maleic acid anhydride, carboxymethyl cellulose, paraffin, vaseline, silica, menthol, azorubin, p-hydroxy-benzoic acid methyl ester. – Does not contain zinc.

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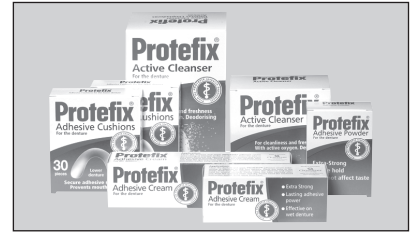
Made in Germany

GB

Firm hold for dentures with Protefix®

Dentures should not prevent you from living life to the full, because this is a justifiable wish of all denture-wearers. There is almost always a solution to denture-wearers' problems. Such problems can occur, for example, when too little saliva is being produced or in the presence of jaw problems. The same applies to temporary or new dentures. As a result, the denture does not fit properly and is painful to wear. Not only that, but you also feel ill at ease in company. **Protefix®** Adhesives provide you with quick and reliable remedies to your denture problems.

The complete range of adhesive and care products for dentures:

- Protefix®** Adhesive Cushion for Upper and Lower Dentures for jaw problems and interim interfacing, prevents sores, provides firm denture hold in cases of severe jaw degeneration, with wet adhesion
- Protefix®** Adhesive Cream Extra-Strong with wet adhesion and long-lasting effectiveness, can be applied straight to the wet denture, when a large amount of saliva is being produced, tasteless
- Protefix®** Adhesive Powder Extra-Strong 100 % natural sodium alginate, provides firm hold when too little saliva is being produced, with wet adhesion, tasteless
- Protefix®** Active Cleanser with Active Oxygen High cleaning power. Prevents bad breath. Freshness and cleanliness all day long.

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